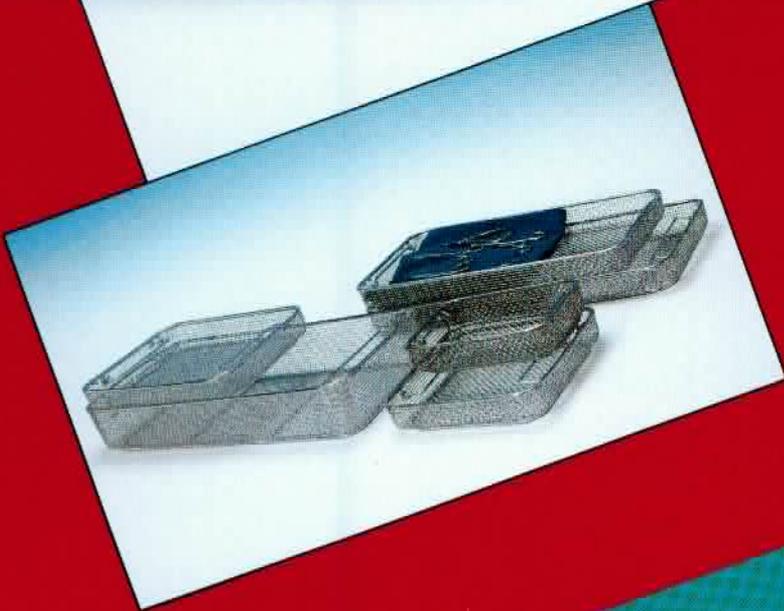
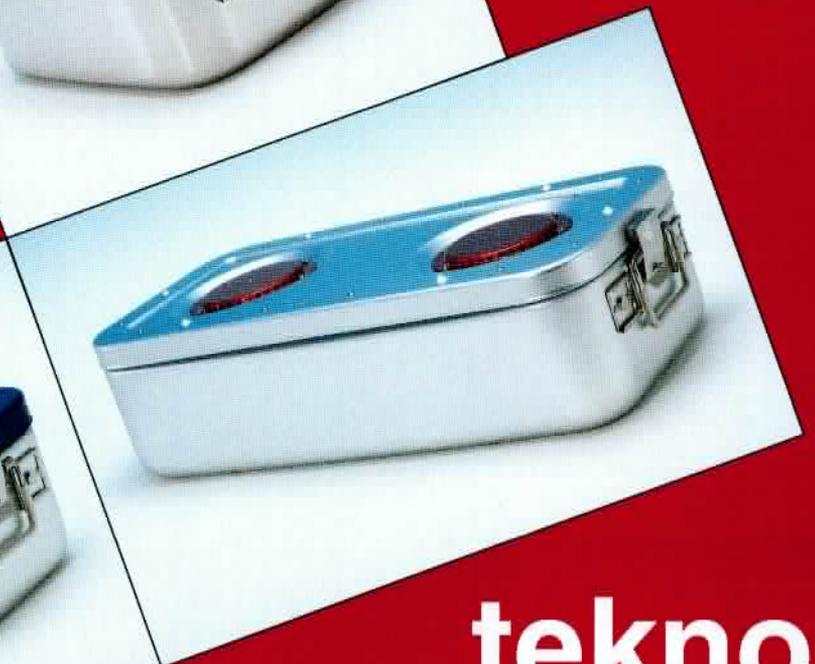
teknomedical

teknomedical
Container
System

Tekno-Medical Optik Chirurgie GmbH&Co.KG

Sattlerstr. 11

D-78532 Tuttlingen

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Sale and

Manufacture of: *Surgical instruments
Dental instruments*

Service:

*Total concepts for Operating theatres
Custom Instruments and Repair
Specialized Sourcing*

Distribution:

- Through specialized dealers in over 50 countries world wide

Our dealers are supported by well trained area managers with excellent language ability. Languages such as; English, French, Spanish, Italian, German and Portugiese, are not only written but fluently spoken.

A vast selection of practical catalog material with over 10.000 different instruments to choose from, makes working with Tekno-Medical easy for both our customers and suppliers.

Fast and efficient customer and supplier order processing is supported by a modern computer system, warehouse, and a motivated team of workers.

Quality:

- has first priority.

Strict Quality Control during all manufacturing phases ensures that "Quality is produced", and before we put our trade mark on, a final and complete inspection must be passed. This is how we guarantee our products. Our Company is certified according to DIN ISO 9001.

*Our **trade name "Tekno"** is registered.*

Manufacturing:

Under the specialized eye of guild certified master craftsmen, our school trained instrument makers, assure that the best german instrument steel is formed into a long lasting, precision product.

The latest knowledge along with state of the art computer controlled machinery ensure that the current norms are not only met, but exceeded.

With the advent of Minimal Invasive Surgery Tekno-Medical was able to keep pace with the rapid changes in this surgical area. Flexibility, documented quality, and innovation today are still the key factors for our success.

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Materials and Processing

In the manufacture of tekno containers both stainless steel (X5 Cr Ni 18/9-10), with it's characteristics of hardness and resistance to oxidation, and aluminum (5005 alloy), with it's excellent heat dissipation capacity along with light weight are employed. While stainless steel does not require any additional surface protection, aluminum by it's nature must receive a protective layer through anodizing. Besides the classical anodizing process which can be coloured for quick identification, Tekno also uses a hard anodizing process. This electrolytic process produces aluminum oxide on the surface giving it a higher resistance level. Hard anodizing is used for items which are subject to heavy stress and use, such as pistons and firearms. The surface colour darkens in proportion to the oxide

thickness to it's unique burnished appearance. Thanks to this treatment the excellent heat dissipation and light weight characteristics remain unchanged, while improving the mechanical. The main parts of the Tekno containers are formed in one piece, so that welding is avoided. The process of welding always puts a thermal strain on the materials and alters the internal structure of the alloy, changing it's characteristics in the weld area. For these reasons the assembly of components is carried out by riveting with stainless steel rivits. All gaskets are of non-toxic silicon, and if the need should arise, are available as replacements. Disposable and reusable-textile filters are available.

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General characteristics

All tekno containers have a rolled edge on the pan which adds to the stability as well as providing a better contact surface for the seal. The security of the lid seal is ensured by the non-toxic expanded silicon gasket which is easy to inspect, clean, and replace if worn. The stacking capability which optimizes the employment of the sterilizing chamber and helps inner handling, has been

planned in consideration of the strain the lid gasket is subject to. In fact the pressure it is already subject to will be increased - due to the weight of the containers stacked upon it - only by few decimillimetres i.e. until the lid leans on the proper mechanical block, safeguarding the elastic return of the gasket itself.

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Aluminum container lids with colour anodizing are supplied, by choice, in the described colours.

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In case of order please specify the number of the required filters: OF, 1F, 2F. Moreover specify if the lid is to be with or without hinges: M/S or O/S

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In case of order please specify the number of the required filters: 1F or 2F. And if the lid is to be with or without hinges: M/S or O/S.

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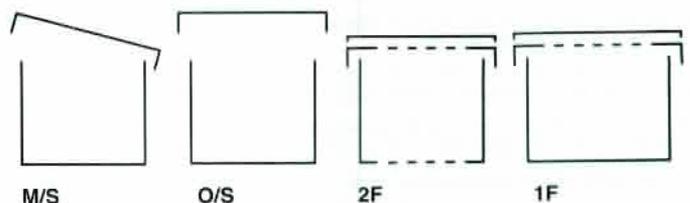
Besides the characteristics of the filtering system, which are the same as the Standard Model Container, the Protected Filter Model Container is provided with a double lid. The inlet of steam and air occurs horizontally on the lid sides, through a tortuous run which prevents dust from depositing on the filter. The vent position does not allow pointed and sharp objects to get to the filter, damaging it and affecting the tightness and sterility of the load.

The double lid avoids that condensate from the containers piled up on it, comes into contact with the filter which, getting wet, could lose its characteristics as microbic barrier.

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The Protected Valve Container is a mechanical filtering system requiring no maintenance.

The function of the valve can be tested by inserting a pointed probe into the hole provided in the center of each valve. The valve's three working positions during and after sterilization are shown in the drawing.

1 Before sterilization the valve is in state of rest.

2 During the vacuum stage depression makes the valve open completely, so that air escapes from the container.

3 Afterwards the valve closes, and the second valve opens; in this stage steam enters and the sterilization stage starts.

1 At the end of the cycle the valve closes air-tight in order to maintain the container load sterile.

"Mini Container Filter" und Körbe "Mini container Filter" and stainless steel baskets

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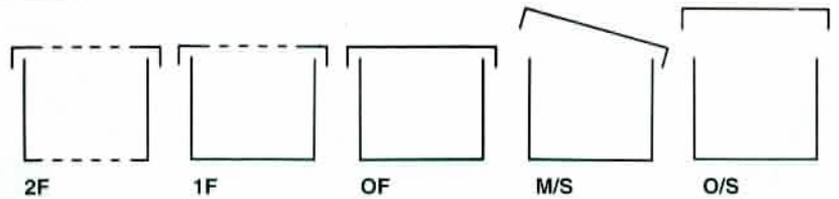
Sterilization baskets - Stainless steel

Art.		L	B	H
86080-25	mm	590	280	255
86081-13	mm	590	280	135
86082-25	mm	280	280	255
86083-13	mm	280	280	135

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Disposable paper label with
chemical indicator

86500-60
Siegel
Seal

86500-70
Tuchhalterklemme
Cloth fixing pin

86500-71
Tuchhalterklemme doppelt
Twin socket cloth fixing pin

86501-10
Tuchhalterahmen 600x300

86501-12
Tuchhalterahmen 460x300

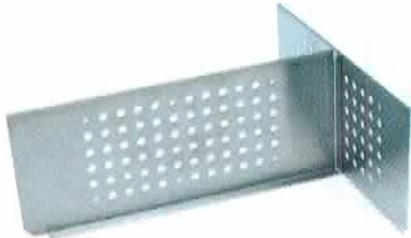
86501-11
Tuchhalterahmen 300x300

86501-10
Cloth fixing frame for module
600x300

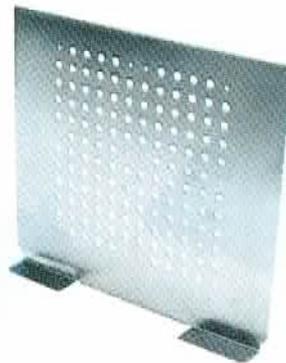
86501-12
Cloth fixing frame for module
460x300

86501-11
Cloth fixing frame for module
300x300

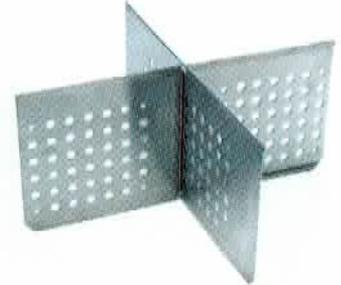
Container-Trennbleche
Partitions for Containers

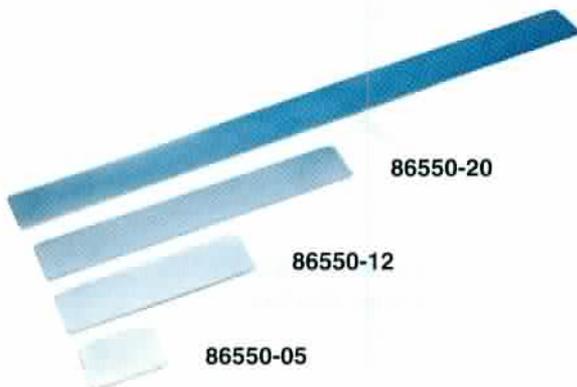
Höhe mm	Container 600x300	Container 300x300
210	86510-21	86511-21
160	86510-16	86511-16
	460x300	
210	86512-21	
160	86512-16	

Container 600x300	Container 300x300
86520-21	86521-21
86520-16	86521-16

Container 600x300	Container 300x300
86530-21	86531-21
86530-16	86531-16
460x300	
86532-21	
86532-16	

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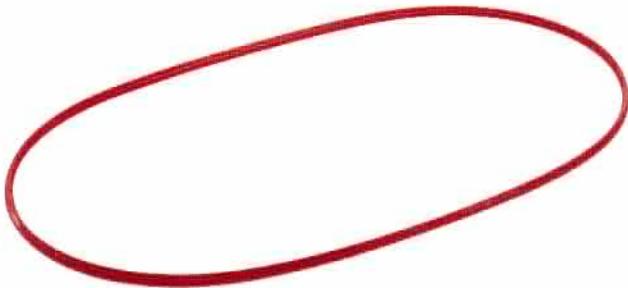
Synthetic fabric filters for "Container Filter", "Protected Filter", "Mini Containers"

Art.	
86580-60	Container 600 x 300
86580-30	Container 300 x 300
86580-30	Container 460 x 300
86580-26	Container 260 x 170

Papier-Filter für "Container Filter", "Geschützter Filter", "Mini Container"

Paper filters for "Container Filter", "Protected Filter", "Mini Containers"

Art.	
86581-60	Container 600 x 300
86581-30	Container 300 x 300
86581-30	Container 460 x 300
86581-26	Container 260 x 170

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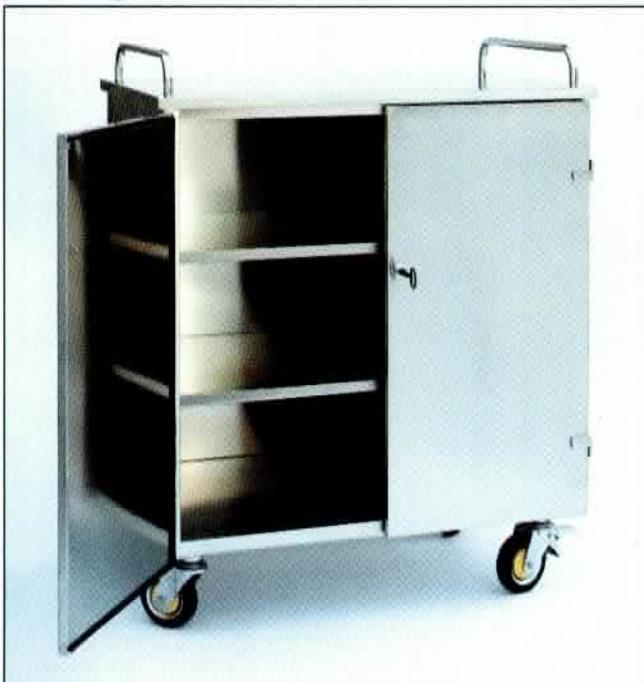
Stainless steel trolley with antistatic wheels and 5 shelves

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Stainless steel trolley with antistatic wheels and 3 shelves

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Tight stainless steel cabinet-trolley, with antistatic wheels. For 9 modules (600x300x300)

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Tight stainless steel cabinet-trolley, with antistatic wheels. For 6 modules (600x300x300)

tekno®

tekno-medical
Optik-Chirurgie GmbH & Co. KG

Sattlerstrasse 11
D-78532 Tuttlingen · Germany

Telefon 0 74 61 / 17 01 - 0
Telefax 0 74 61 / 17 01 50
e-mail: mail@tekno-medical.com